

Episealer[®] Knee Implant - 24-Months' Follow-Up*

*Data presented at the ICRS 2019 World Congress (Vancouver, Canada), October 2019.


PATIENTS

- 86 patients at 12 months' follow-up (62 patients at 24 months)
- Average age at surgery: 49 years (range 27-69)
- Medial condyle: 69 - lateral condyle: 6 - trochlea: 11

RESULTS & CONCLUSIONS

- All KOOS subdomains show statistically significant improvements ($p < 0.005$)
- Rapid pain relief demonstrated within 3 months of surgery
- Good clinical improvement

Mean KOOS** scores preoperatively and at 3, 12 and 24 months' follow-up

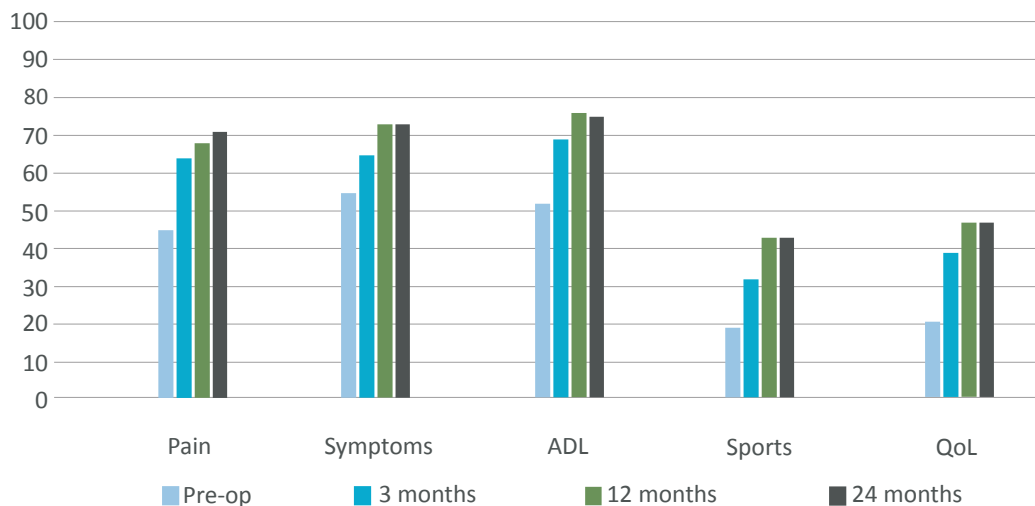

**Knee Injury and Osteoarthritis Outcome Score

